



quadrant

biosciences

Interpretation of Results User's Guide

Purpose: The intended purpose of this document is to guide the user on how to interpret and manage patient results. This guide is for reference only and should not be the sole determination for patient management. Please refer to a primary care provider for explicit direction.

OM View Result	Interpretation	Clinical Significance
Pooled Presumed Negative	Patient was present in a pool that is negative for SARS-CoV-2	Patient is considered presumed negative for SARS-CoV-2
Pooled Positive	Patient was present in a pool that was positive for SARS-CoV-2. Each patient in the pool will be retested individually. Expected turnaround time for this testing is 12-24 hours.	All patients in a positive pool are presumed positive until the reflex testing is completed. Each patient should practice safety measures per institution's precautionary measures.
Reflex Positive	Patient was present in a pool that was positive for SARS-CoV-2. Patient sample was retested individually and identified as positive for SARS-CoV-2.	Patient verified to be positive for SARS-CoV-2.
Reflex Negative	Patient was present in a pool that was positive for SARS-CoV-2. Patient was retested individually and identified as negative for SARS-CoV-2.	Patient verified to be negative for SARS-CoV-2.



quadrant

biosciences

Results continued...

OM View Result	Interpretation	Clinical Significance
Indeterminate	Patient was present in a pool that was Indeterminate for SARS-CoV-2. The virus was detected in the sample but under the quantification levels of the assay. Each patient in the pool will be retested individually. Expected turnaround time for this testing is 12-24 hours.	All patients in an indeterminate pool should be monitored as per institutions precautionary measures. Indeterminate results usually indicate low viral load and can indicate either the start or end of an infection.
Test in Progress	Patient sample is in the lab and is in process.	Turnaround time is approximately 24-48 hours after arrival in the laboratory.
Sample Requires Recollection	Patient sample unsuitable for testing.	Please submit another sample as soon as possible.
Cancelled	Specimen was received beyond stability (e.g. more than 6 days since collection, not enough volume of stabilizer in collection tube, an exception is not addressed within the 6 day stability window).	Test cannot be processed. Please submit another sample as soon as possible.
In Transit	Patient sample has been registered in the app but has not arrived at the lab.	Turnaround time is approximately 24-48 hours after arrival in the laboratory.